


# Blinatumomab Backpack Guide for Use

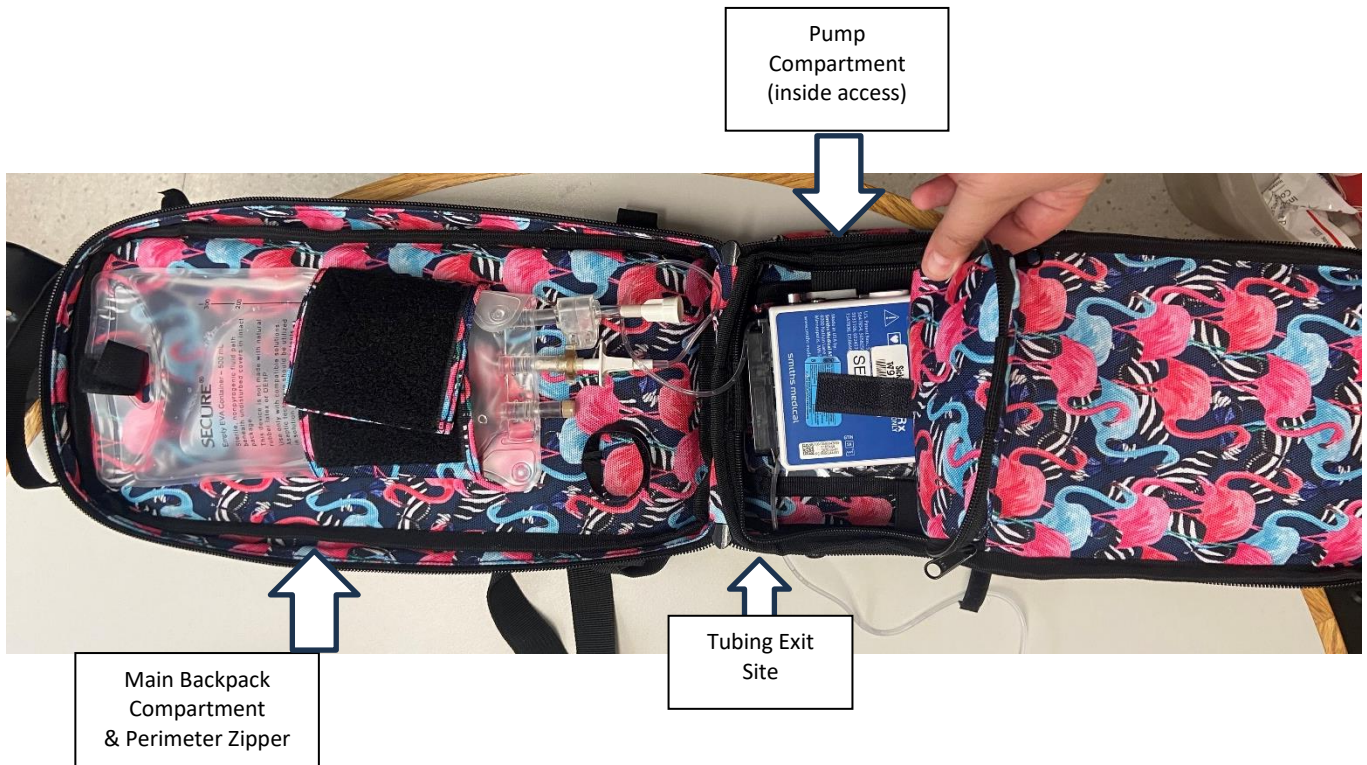

## Blinatumomab Backpack Guide for Use

1. Load medication bag and pump into backpack **after** it has been primed down in the biosafety cabinet.

2. Unzip large zipper around the perimeter of the backpack, open backpack, and lay flat.

3. Using the velcro ties through the top of the medication bag, and around the bag, secure medication bag into main backpack compartment.



4. Unzip the pump compartment inside the backpack.


5. Lay the pump, screen facing down, into this compartment.



6. Feed the end of the tubing through the side zipper in the pump compartment (tubing exit site).

7. Zip the pump compartment closed.

\*Take care that the tubing sits along the covered portions of the zipper.

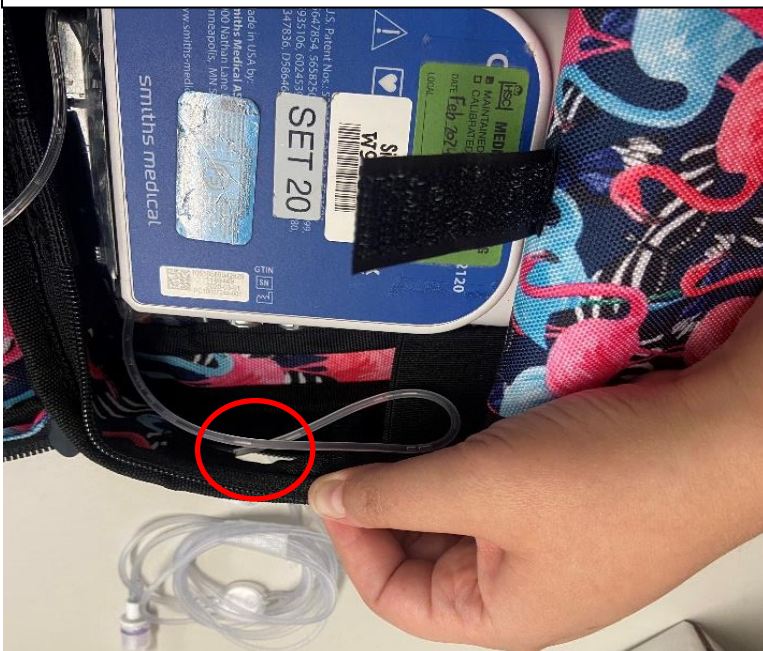


8. Close the large zipper around the perimeter of the backpack.

9. Full medication bag, pump, and tubing should be secured inside the zipped backpack. The only place where tubing should be exposed is through the side zipper in pump compartment.


10. Connect tubing to patient, and coil any extra tubing using securement strap on outside of bag.



## Blinatumomab Backpack Guide for Use

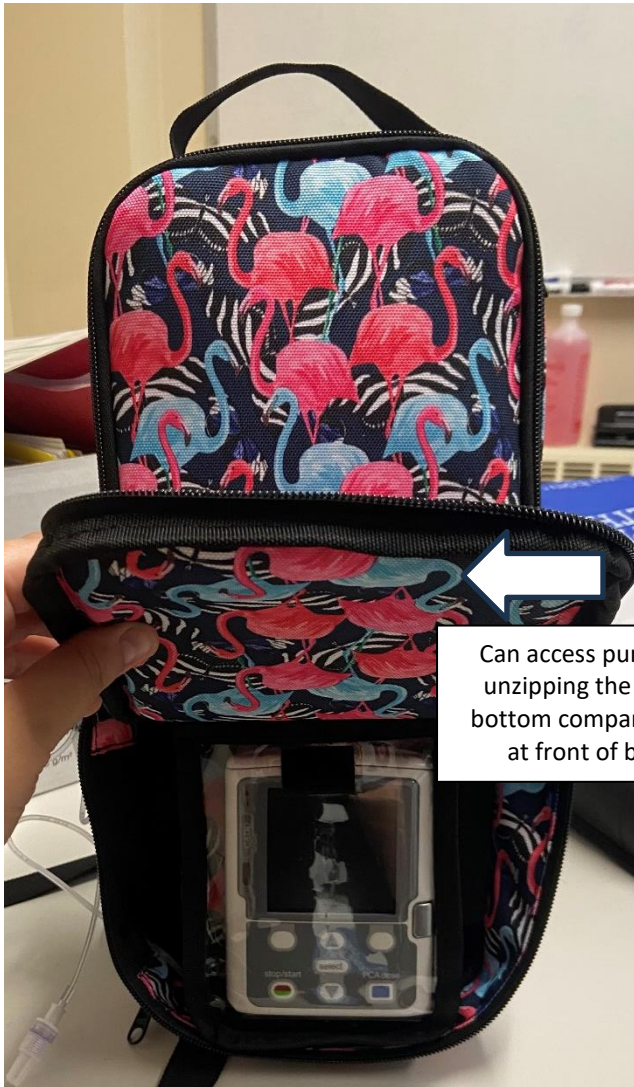

Can access pump by unzipping the small bottom compartment at front of bag.

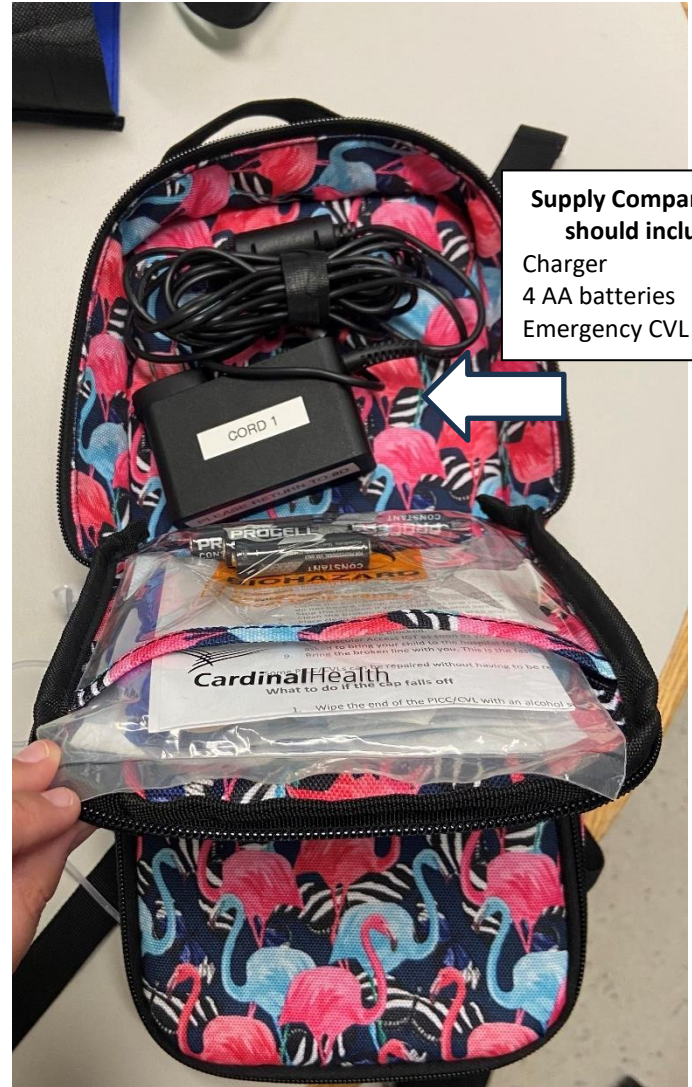

**Supply Compartment should include:**

Charger  
4 AA batteries  
Emergency CVL Kit